

Steps For Disinfecting Telehealth Equipment and Peripheral Devices

Note: This poster appears as Table 1 in the IPC [Best Practice Recommendation for Bedside Computers and Electronics](#). Patient environment means the area immediately surrounding the patient (within two (2) metres). If you have any questions or comments contact IPC at [IPC Surv&Std Admin](#).



1

PERFORM HAND HYGIENE

A



B



Perform hand hygiene with an alcohol-based hand rub (A) or soap and water (B).

Touch panels, remote controls and peripherals will be cleaned before and after each use with these four steps.

2

USE READY-TO-USE LOW LEVEL DISINFECTANT



Remove wipe, close lid tightly and squeeze wipe to remove excess fluid. Follow manufacturer label for glove use.

3

DISINFECT



Use one side of the wipe to clean the surface. Use the other side of the wipe to disinfect, discard. Allow surface to fully air dry.

4

PERFORM HAND HYGIENE

A



B



For more information contact
ipcsurvstdadmin@ahs.ca
© 2023 Alberta Health Services, IPC

Version	Date (YYYY-MM-DD)
Created	2016-07-18
Updated	2018-10-10
Revised	2023-12-07